

Sample Collecting Cylinders




This product is a sample collection device developed by Nippi Inc. for BSE testing. It allows for easier collection of samples weighing $110 \pm 20\text{mg}$ from the medulla oblongata of cattle.



Packaging: 100 pieces /bag
Material: PP (Polypropylene)

Instructions for use: Lower the black piston to create a gap of approximately 3mm at the tip of the cylinder (Fig. 1). Use the tip of the cylinder to cut the sample from the target area. Verify that the sample weighs $110 \pm 20\text{mg}$ using a balance. Insert the sample into the collection tube and push down the piston (Fig. 2).

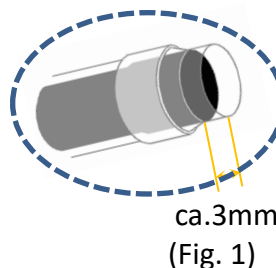

Product code	Product name	Packaging	List Price
893503	Sample Collecting Cylinders	100 pcs/bag	JPY36,000

Nippi, Incorporated Biological and Chemical Product Division

1-1-1, Senju Midori-cho, Adachi-ku, Tokyo 120-8601, JAPAN TEL:03-3888-5184 FAX:03-3888-5136

